

Pre-flight Checklist for Manikin-based Scenarios using AV Control

- Step 1: Open the SIMULATIONiQ AV Control application.
- Step 2: Review Health Banner for any issues.
 - If any red or yellow icons, review the message.
 - If you are given directions how to fix it, go to the System Monitor to make the changes.
 - If you're not comfortable making the changes, call EMS TotalCAREiQ team.
- Step 3: Prepare your simulator(s) and vitals monitor(s.)
- Step 4: Using AV Control to see that the cameras are positioned correctly.
- Step 5: Start a recording (at least 2 mins. long) in all the room(s) that will be used today. (Be sure that there is some type of audio in the room during recording.)
- Step 6: Click on AV Viewer, search for recording and play it (with audio.)
- Step 7: Start the simulator scenario. (If you are set up for automatic recording from the simulator's instructor laptop, then disregard Step 8.)
- Step 8: Start recording in AV Control.

If you have any problems completing this checklist, contact EMS TotalCAREiQ team at csupport@simulationiq.com or (800) 405- 7500.